arexim

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Regulation (EU) 2016 / 425

The declaration of conforming can be found on the website <https://www.areximengineering.bg/en/products/>

Protective face shield, model ARX-S, in accordance with the essential requirements of Regulation (EU) 2016/425 and with the requirements of EN 166: 2001
Purpose: Protective face shield, model ARX-S, is designed for protection against drops and splashes of liquid (sneezing, saliva, aerosols). Reduces the risk of infection by splashes and droplets. Due to its design, full protection against viruses, bacteria or biological agents is only available to a limited extent.

Caution !!! The protective face shield offer no protection against mechanical hazards.

Intensive liquid stream (showers, washing installations, etc.) may allow liquid to penetrate.

Caution !!! In the event of exposure to steam or abrupt temperature changes, the glass may become blurred.

Marking
Identification of manufacturer.....AREXIM

Code	Designation	Description of the field of use
3	Liquids	Liquids (droplets or splashes)
4	Large dust particles	Dust with a particle size of > 5µm
5	Gas and fine dust particles	Gazes, vapours, sprays, smoke and dust with a particle size < 5µm
8	Short circuit electric arc	Electrical arc due to a short circuit in electrical equipment
9	Molten metal and hot solid	Splashes of molten metal and penetration of hot solids

Areas of application: Sterile working environments (laboratories, hospitals, cleanrooms); production (food industry, etc.), services (cleaning, painting, etc.), animal husbandry.

Please read carefully:

Under normal operating conditions, the duration of use is not more than 6 months. For intensive (daily or continuous) use, the period of use should not exceed 25 working days.

Combination with other protective equipment: Protective face shield, model ARX-S can be used with diopter glasses and also in combination with a Protective goggles - ARX or similar and with protective masks.

Storage and transport: Store in dry areas at 5°C ÷ 40°C and away from direct sunlight. Drop shipping and loading are not offered - spoiled activity, whatever is placed on the heavy objects of these.
Disposal: The protective face shield ARX - S model should be disposed after expiration of the recommended period of use. A reuse of the face shield is at the sole responsibility of the user.

Caution !!! The protective face shield must be disinfected before disposal!

Warranty Terms - Arexim Engineering EAD is responsible for manufacturing defects throughout the product's lifetime. The warranty does not apply in the presence of traces resulting from normal use - discoloration, injury, scratches, etc. that is not a manufacturing defect.

Caution !!! If the protective face shield is damaged or broken, must be replaced, as they do not guarantee your protection. For spare parts please contact us or your distributor.

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HOW TO PUT FACE SHIELD PROPERLY
Put the hold frame in the central gap of the shield (fig.1)
and after that rotate the frame to 90 degrees (fig.2).
Put the rest two ribs on the left and right side in the "T" profile of the visor
and then push the frame in the direction shown on the picture to make "click" (fig.3).
Adjust the length of the head band and pin on (fig.4).
Put the shield on your head (fig.5) and pull the shield to your face (fig.6).
When removing and disassembling the shield, see (fig. 7) and (fig. 8)

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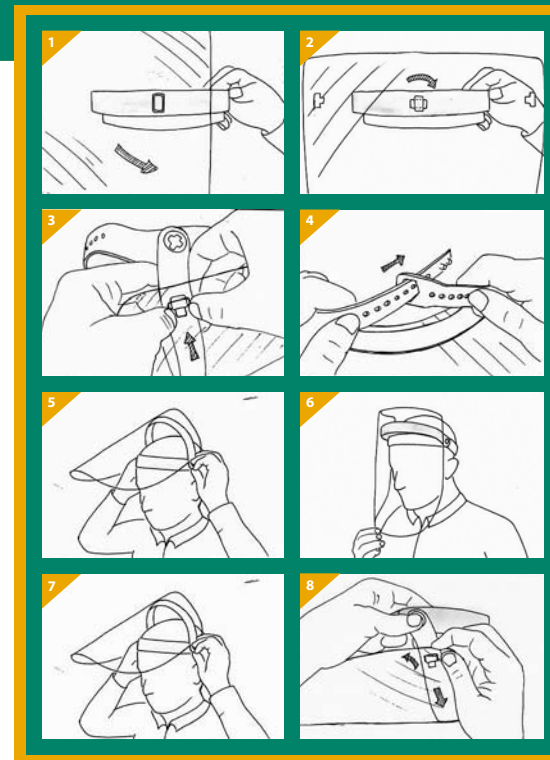
Cleaning and disinfection: The protective face shield is reusable!
Cleaning and disinfection after each use is recommended,
If the shield is used continuously, disinfection on each 4 hours is recommended!
Cleaning: The cleaning of the protective shield is done with a cotton microfiber towel,
and a detergent (soap, etc.). Cleaning with dry towel is not allowed.
Disinfection: It is important to protect yourself before cleaning your shield. Use gloves, face mask
and, if possible, other goggles / face shield. Disinfect the area where you will be cleaning. Do not
treat several shields at a time, split them into several batches, avoid cross contamination! To ensure
that your shield is properly cleaned, we recommend to disassemble it (fig.8).
It is highly recommended to immerse (submerge) the shields in disinfectant, thereby ensuring that
the solution reaches 100% of the surface. You can also use aerosol (spray), but apply the procedure
at least twice to reach most of the surface.
You can remove debris from the disinfectant with a paper towel, avoid using the same towel on
multiple shields, avoid cross contamination. Always wipe in one direction.

METHOD	CONDITIONS
Hot Air Dryer	65 °C (149 °F), 60 mins
Isopropanol (IPA)	75 - 95%, 5 mins
Ethanol	70 - 80%, 5 mins
Hydrogen Peroxide	6 - 10%, 5 mins
Soap water	5 mins

Caution !!! Cleaning with chlorine or abrasive-containing preparations is not allowed!, UV
sterilization, gas-plasma sterilization and sterilization at temperatures higher than 70 °C is not allowed
as well.

Caution !!! The product is not pre-sterilized! Clean and disinfect before use!

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